TuBo-Fix

non-adhesive

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suitable also for double lumen tubes and larynx-masks

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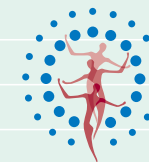
www.salubrious.se

TuBo-Fix

Place the padding around the patient's neck with the **white** cotton surface facing the skin. If necessary, cut the excess part off.

Stick the adhesive tape on the inside of the middle part or around the tube. The tube has to be free of secretion, dirt and fat. Take the cover off.

possibility A:
Cross the middle part once above the tube.

possibility B:
Loop the strip via the perforation.

Fix the middle part with a slight tug on the padding using the Velcro® strip. Readjustment is possible at any time and without difficulty.

Completed fixation
In case of dirt, the middle part can be replaced separately. The padding might stay on the patient for a longer period of time.

Please visit our homepage www.novo-med-group.com for further modes of application

TuBo-Fix Application with Guedel Airway

Place the padding around the patient's neck with the **white** cotton surface facing the skin. If necessary, cut the excess part off.

Stick the adhesive tape on the inside of the middle part or around the tube. The tube has to be free of secretion, dirt and fat. Take the cover off.

possibility A:
Cross the middle part once above the tube.

possibility B:
Loop the strip via the perforation.
Insert the guedel airway.

Fix one end of the middle part on the padding using the Velcro® strip. Wrap the other end of the middle part around the guedel airway and fix it with a slight tug on the padding.

Completed fixation. In case of dirt, the middle part can be replaced separately. The padding might stay on the patient for a longer period of time.