



REPAIR KIT

*A repair kit for
GelCell and Aerogel
pressure relieving products*

GelCell



Aerogel



Instructions for Use

1. Ensure that the surface of the pad to be repaired is both clean and dry.
2. Cut the patch from the adhesive sheet to the size required, allowing at least 2cm ($\frac{3}{4}$ ") border around the damaged area.
3. Remove the backing paper from the adhesive sheet and apply it to the area to be repaired.
4. Apply firm pressure to the patch to ensure effective adhesion.

This pack contains adhesive coated polyurethane film.

TL/PC/089/0201



Manufactured by
TRULIFE

Cookstown Industrial Estate, Belgard Road, Tallaght, Dublin, 24 Ireland.
Tel: +353 | 45 | 1755 Fax: +353 | 4525790 Web: www.trulife.ie

